

Product line:

Healthcare


plastic

**Use and
maintenance
manual**

Splash-resistant
face shields

**Face shield for
small heads**

Ref. **CHILDREN**

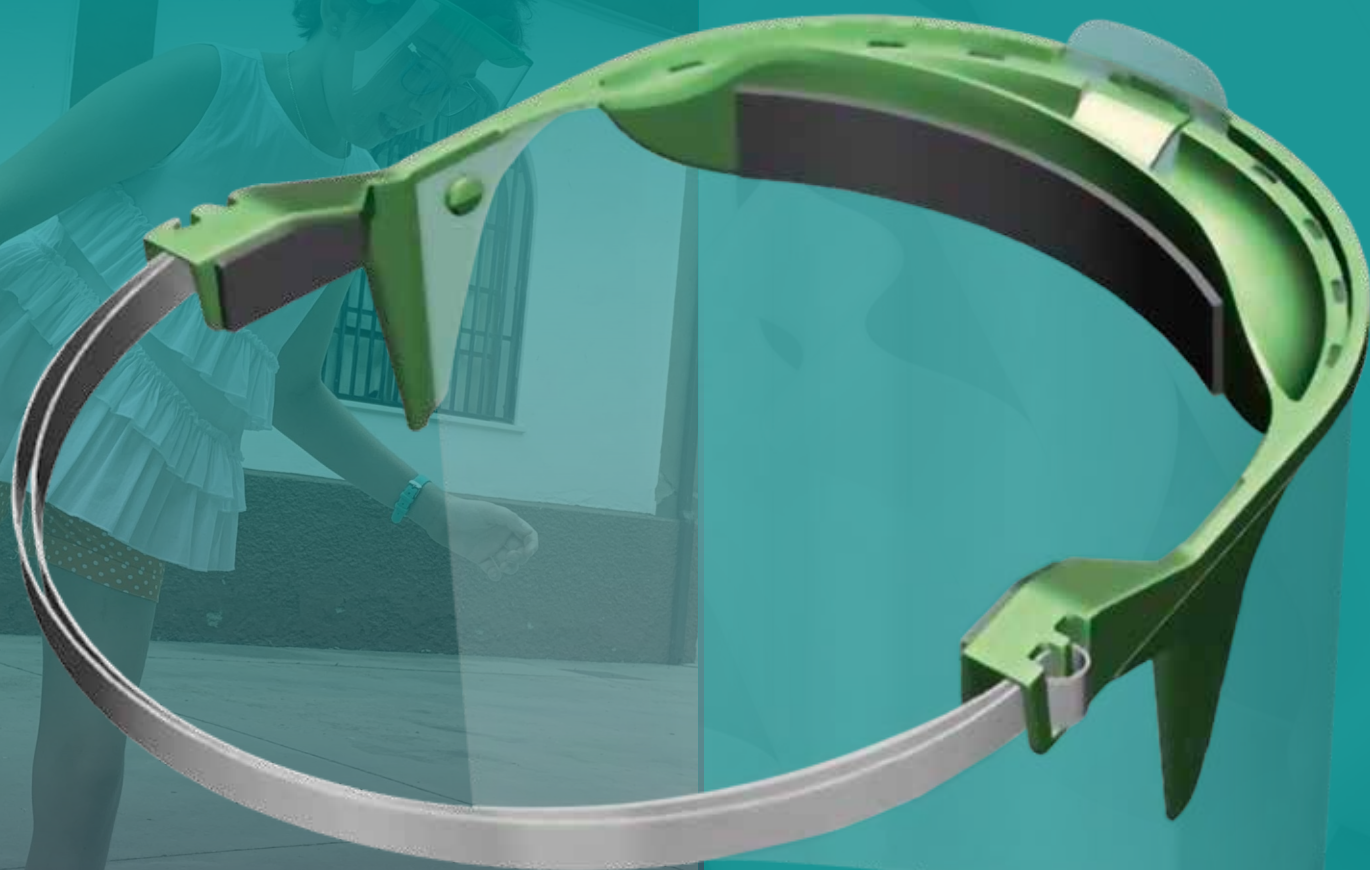

RFU 03.031/V1 - INYEC & CEP, S. L. -1-3

CAT II

Pol. Ind. La Vega. C/ El roble, 8.
C.P.: 14640 - Villa del Río. Córdoba (España)
Telf.: (+34) 957 176 352
gestion@inyeccep.com

www.inyeccep.com

INDEX

1. INTRODUCTION

2. USE

2.1. CLEANING RECOMMENDATIONS

2.2. MAINTENANCE

2.3. STORAGE

2.4. PACKAGING

2.5. APPLICATION SCOPES

3. MARKING AND FEATURES

4. USE RESTRICTIONS

5. PLACEMENT AND ADJUSTING

6. WARNINGS

7. DECLARATION OF CONFORMITY



1. INTRODUCTION

Manual de uso y mantenimiento CHILDREN

This manual is intended to provide the customer with guidelines for operation, maintenance and instructions on the use of this product supplied by the manufacturer.

This product has been manufactured following the requirements of Regulation (EU) 2016/425, for basic use, according to the Recommendation for Use RfU PPE-R/03. 031 Version 1 on the basis of Commission Recommendation (EU) 2020/403 of 13 March 2020 on conformity assessment and market surveillance procedures in the context of the threat posed by COVID-19, to protect against drops and splashes, which are applicable to it, as stated in certificate No. 20/3464/00/0161 of AITEX, Plaza Emilio Sala No. 1, Alcoi, Spain, Notified Body 0161.

This PPE contains the following materials:

- **Visor:** polypropylene.
- **Screens:** polyethylene terephthalate polyester.
- **Adhesive protectors:** Polyurethane foam.
- **Adjustment device:** elastic rubber band 12mm braided seam.



2. USE

The face shield is placed on the user's head by adjusting the elastic band until it is perfectly positioned, leaving the hands free to perform operations.

2.1. CLEANING RECOMMENDATIONS

Two different types of disinfectant can be used, with their corresponding proportions. All disinfectant solutions must be water-based.

- Disinfection with bleach: Prepare a solution of nine parts water to one part bleach.
- Disinfection with alcohol: Prepare a solution of three parts water to seven parts alcohol.

2.2. MAINTENANCE

To protect the eyepiece

- Do not attempt to disassemble or force the equipment, as proper protection of the equipment depends on it.
- The user must not modify the design or configuration of the equipment.
- Keep the equipment clean before use to ensure proper operation.
- After use, the equipment should be stored in places protected from moisture and radiation.
- This product should be kept away from heat sources such as open flames, radiators, ovens, etc.

2.3. STORAGE

- Each eyepiece is packed in a bag that protects it from dust and moisture.
- Replace in the protective bag when not in use.
- Store between 5 and 40°C, with humidity less than 90%.

2.4. PACKAGING

- Each unit supplied consists of one frame, two shields, two sets of adhesive protectors and two adjusting devices.
- Supplied in carton boxes of 25 units.

2.5. APPLICATION SCOPES

Intended for eye and face protection in activities where droplet and splash protection is required during COVID-19 emergency only. This face shield is intended for drop and splash protection during the COVID-19 emergency only and is not intended for general use, nor should it be used for any other purpose.



3. MARKING AND FEATURES

The **CHILDREN** PPE shall be marked:

- **RfU 03.031/V1-INYEC & CEP, S.L.-1-3**

This marking refers to:

RfU 03.031/V1: that it has been certified according to the Recommendation for Use RfU PPE-R/03.031 Version 1 on the basis of Commission Recommendation (EU) 2020/403 of 13 March 2020 on conformity assessment and market surveillance procedures in the context of the threat posed by COVID-19.

INYEC & CEP, S.L.: identification of the fabricant

- 1: Optical Class 1
- 3: Protection against droplets or splashes of liquid

4. USE RESTRICTIONS

- This personal protective equipment is designed solely and exclusively for the function specified by the manufacturer, i.e., to be used as splash protection equipment.

The use of this type of shields is exclusively for protection against non-corrosive and non-incandescent splashes. These shields have been designed for the protection of the user against particle protection sputtered by a patient infected with a virus that can be transmitted by coughing, sneezing, saliva, etc. such as the COVID-19 coronavirus.

Thus, this face shield **does not protect** against mechanical impact, optical radiation, molten metals, dust, gases, or electric arcs. **The PPE does not protect against aerosols or particles in suspension.**



Product line:

Healthcare plastic

"Children" Face shield

Face shield for children
between 3 and 17 years of age



CAT II

RfU 03.031/V1 - INYEC & CEP, S. L. -1-3

Pol. Ind. La Vega. C/ El Roble, 8.
C.P.: 14640 - Villa del Río. Córdoba.
Telf.: (+34) 957 176 352

www.inyeccep.com



Washable and
reusable



Recyclable
equipment



Very light
(44 grs.)



Made in
Spain

Product line:

Healthcare plastic

"Children" Face shield

Face shield for children
between 3 and 17 years of age



Washable and
reusable



Recyclable
equipment




Very light
(44 grs.)



Made in
Spain

- **Ergonomic design** with adaptable facial adjustment.
- **Materials:** headband made of **polypropylene** material that facilitates facial adjustment and screen made of **PET** material that offers excellent resistance to impacts, thermal deformation and with a high degree of transparency.
- **They protect** all virus transmission channels and help us **not to touch our face** with our hands unconsciously in moments of exposure.
- **They make it easier to interact** with colleagues and teachers in the educational environment.
- **No need to remove the equipment** to facilitate communication and can be reused indefinitely.
- **Anti-fogging system**, through openings in the headband, prevents condensation.
- **Easy cleaning** with soap and water or common disinfectants.
- **It is recommended to personalize** with the student's name on the headband.
- **Spare parts for this equipment** are provided in sets of 3 pieces at www.inyeccep.com.

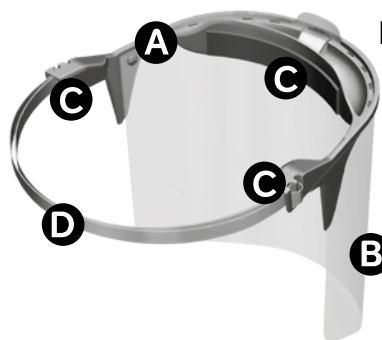


KIT COMPOSED OF:

- **A.** Headband
- **B.** Face shield
- **C.** Foam.
Adhesive protector
against chafing
- **D.** Elastic band

Contains: 1 Headband, 2 Foam, 2 Tapes, 2 Screens,
Instruction guide for use

FOAM PLACEMENT


Adjust the foam
by removing the
adhesive tapes on
the front and sides
of the headband.

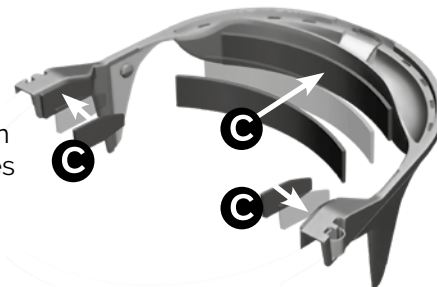

ELASTIC BAND ASSEMBLY


Step 1. Insert the two
ends of the rubber
band.

Step 2. insert the two ends
together, from the inside to the
outside, and once outside, insert
again through the second slot, from
the outside to the inside and knot
them with a loop, pull to tighten.

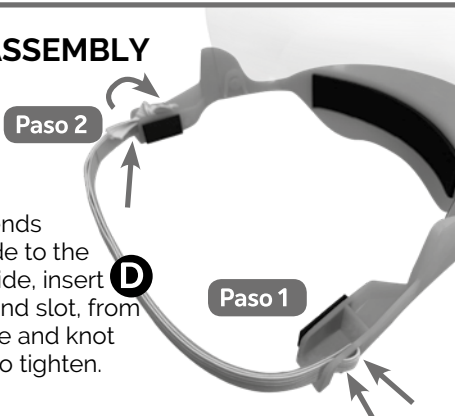

SCREEN ATTACHMENT

Adapt the
screen to the
circular
shape and
clip the side
and front
tabs.

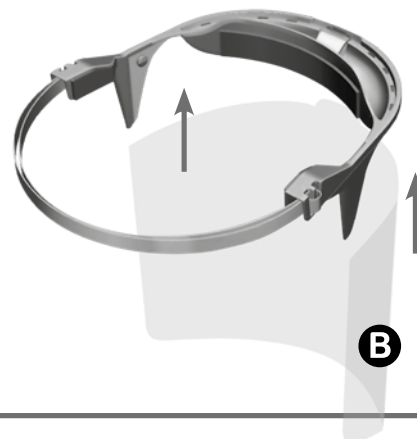

INYE & CEP

Inyección de piezas de plástico

Pol. Ind. La Vega. C/ Roble, 8.
C.P.: 14640 - Villa del Río. Córdoba.
Telf.: (+34) 957 176 352

www.inyeccep.com



RFU 03.031/V1 - INYEC & CEP, S. L. -1-3

CAT II

6. WARNINGS

- Adapt the screen to the circular shape and clip the side and front tabs.
- WARNINGS
- Materials that come into contact with the user's skin may cause allergies in sensitive people.
- Replace if any deterioration of the PPE is observed.
- It is advisable to replace scratched or damaged eyepieces.
- The eye protection device (face shield) does not offer protection against mechanical impacts.
- The eye protection device (face shield) should only be used in health care settings and in activities where protection against drops and splashes is required, and only during the current COVID-19 emergency.
- The eye protection device (face shield) protects against liquid drops and splashes. This reduces the risk of droplet infection.
- **The device can be reused if cleaned according to the manufacturer's instructions.**

THE EPI HAS A SHELF LIFE OF 1 YEAR.

The operating life will also depend on the use of the PPE, maintenance, storage, etc.

Year of manufacture: 2020

7. DECLARATION OF CONFORMITY

DECLARACIÓN UE DE CONFORMIDAD
"EU" DECLARATION OF CONFORMITY
DECLARATION "UE" DE CONFORMITÉ

This declaration of conformity is issued under the sole responsibility of the manufacturer as established by the EC:



Pol. Ind. La Vega. C/ El Roble, 8.
C.P.: 14640 - Villa del Río. Córdoba. (España)
Telf.: (+34) 957 176 352
gestion@inyeccep.com

www.inyeccep.com

Declara que el diseño y fabricación del
PRODUCTO /

Declares that the design and construction of the
PRODUCT /

Déclare que le dessin et construction du produit:

PRODUCT TYPE: FACE SHIELD

BRAND NAME: HEALTHCAREplastic

MODEL: CHILDREN

INDIVIDUAL PROTECTION EQUIPMENT CATEGORY: Category II

It conforms to the provisions of EU Regulation 2016/425 of 9 March 2016 and, in particular, the specifications of the Recommendation for Use RfU **PPE-r/03.031 Version 1** on the basis of Commission Recommendation (EU) 2020/403 of 13 March 2020 on conformity assessment and market surveillance procedures in the context of the threat posed by COVID-19 to protect against drops and splashes.

The notified body **ASOCIACIÓN DE INVESTIGACIÓN DE LA INDUSTRIA TEXTIL - AITEX**, notified body no. 0161 at Plaza Emilio Sala 1, 03801 Alcoy, Alicante, has carried out the EU type examination certificate (module B) and issued EU type certificate no. **20/3464/00/0161**.

DATE OF ISSUE OF THIS DECLARATION OF CONFORMITY: 24/11/2020

FIRMA: Juan Muñoz Anguita, Gerente.